

# NANO

MED-CLEAN

## ZDRAVOTNICKÁ MASKA

Filtrační účinnost materiálu je **99,9 % VIRŮ A BAKTERIÍ**.  
Snadné dýchání díky **VYSOKÉ PRODYŠNOSTI 200 l/m<sup>2</sup>/s**.  
Jednorázová. Riziko kontaminace v případě opakovaného použití.



### VYROBENO V ČESKÉ REPUBLICE

**Výrobce:**

Nano Medical s.r.o., 5. května 65, Praha 4 140 21

**Distributor:**

Triola a.s., Mariánské Údolí 1, Horní Jiřetín 435 43



EN 14683+AC

## CZ | NÁVOD K POUŽITÍ

### Jak se nasazuje maska s gumičkami:

1. Umyjte a utřete si ruce a vyjměte masku z krabice.
2. Uchopte vyztuženým ohybatelným okrajem směrem nahoru.
3. Držte masku za poutka a zahákněte poutka za uši.
4. Vyztužený okraj vytvarujte podle tvaru svého nosu. Přetáhněte spodní okraj masky přes ústa a bradu.
5. Na masku během nošení nesahejte.

### Jak se sundává maska s gumičkami:

1. Umyjte a utřete si ruce.
2. Nedotýkejte se přední strany masky, která je kontaminovaná. Dotýkejte se pouze gumiček.
3. Uchopte obě gumičky, opatrně zvedněte a sejměte masku.
4. Vyhodte masku do odpadu.
5. Umyjte si ruce.

Verze návodu: 13. 7. 2020

info@nanomedclean.com  
+420 476 769 977

[www.nanomedclean.com](http://www.nanomedclean.com)  
[www.triola.cz](http://www.triola.cz)

# NANO

## MED-CLEAN



### MADE IN THE CZECH REPUBLIC

**Producer:**

Nano Medical s.r.o., 5. května 65, Praha 4 140 21

**Distributor:**

Triola a.s., Mariánské Údolí 1, Horní Jiřetín 435 43



### EN 14683+AC

## SURGICAL FACE MASK

Filtration efficiency of **99,9 % AGAINST VIRUSES AND BACTERIA.**  
**HIGH BREATHABILITY 200 l/m<sup>2</sup>/s.**

Disposable. Risk of contamination with repeated use.

### ENG | INSTRUCTIONS FOR USE

#### How to put on a face mask with ear loops:

1. Wash and dry your hands with soap and water or hand sanitizer before touching the mask.
2. Remove the mask from the box.
3. The side of the mask that has a stiff bendable edge is the top.
4. Hold the mask by the ear loops. Place a loop around each ear.
5. Mould the stiff edge to the shape of your nose.
6. Pull the bottom of the mask over your mouth and chin.
7. Don't touch the mask while wearing in.

#### How to remove a mask with ear loops:

1. Wash and dry your hands before touching the mask.
2. Avoid touching the front of the mask that can be contaminated. Touch only the ear loops.
3. Hold both of the ear loops and gently lift and remove the mask.
4. Throw the mask in the trash.
5. Clean your hands with soap and water or hand sanitizer.

**Instruction manual version: 13. 7. 2020**

info@nanomedclean.com  
+420 476 769 977

[www.nanomedclean.com](http://www.nanomedclean.com)  
[www.triola.cz](http://www.triola.cz)